
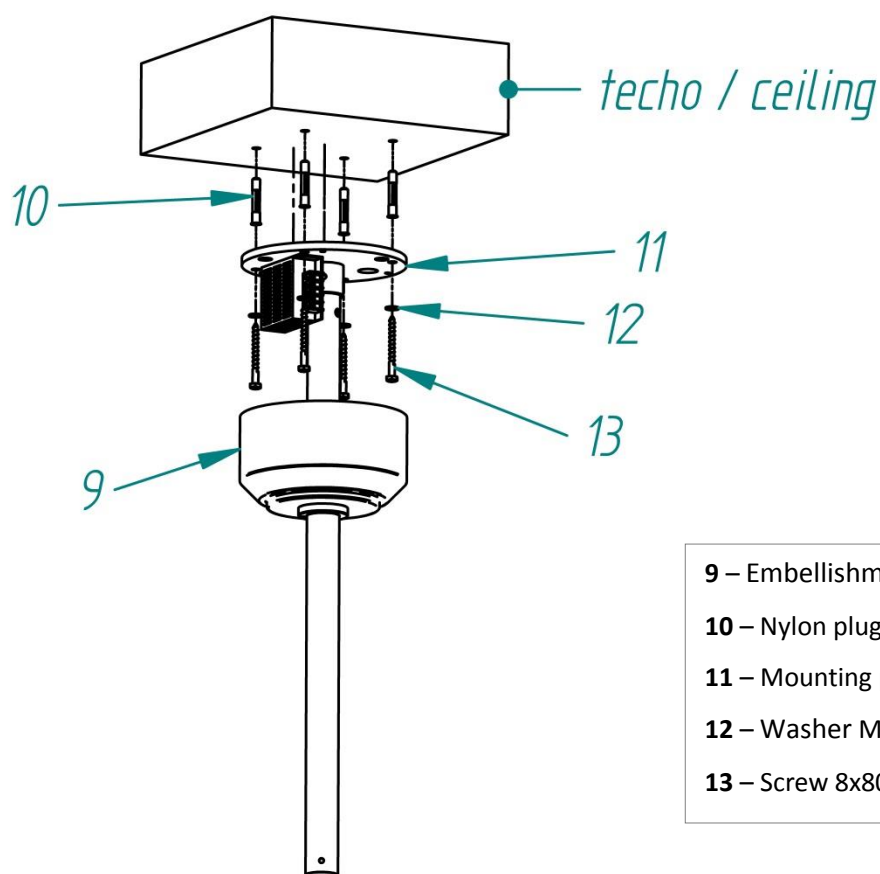


Case C: Anchorage fixed directly to ceiling

This type of fixture is designed for concrete rooftops executed according to DIN 1045. In other types of roof the fixation must be projected in the building work.

For fixing the anchorage to the ceiling please read the following instructions (See picture 1):

1. Use the provided template to mark **four** holes. The holes must be of **Ø10** and a depth of **100mm**.
2. Insert the plugs **#10** into the holes and subject the mounting plate **# 11** by using washers **#12** and screws **#13**. Level the plate **# 11** to avoid further problems.
3. Insert the embellishment (**# 9**) in the vertical arm and place it next to the ceiling.

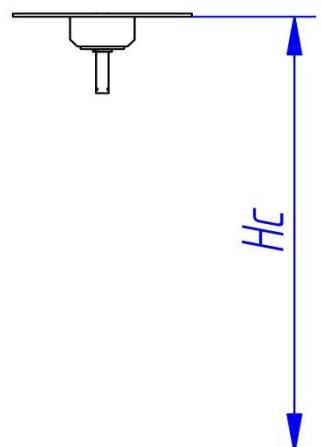

- 9 – Embellishment
- 10 – Nylon plug of Ø10 x 80mm (x4)
- 11 – Mounting plate
- 12 – Washer M8 DIN 9021 (x4)
- 13 – Screw 8x80mm DIN 751 (x4)

Fig. 1- Assembly of the anchorage