

# **INSTRUCTION MANUAL**

DIGITAL THERMOMETER WDT404(Standard) & WDT505(Deluxe)

### Thank you for purchasing this Welcare Digital Thermometer.

Please keep this instruction manual in a safe place where it is readily available. Note: The illustrations in this manual may differ slightly from the product itself



# Read Instructions Carefully Before Use

### Important Safety Information

Read this section thoroughly before using the product. Please ensure you observe the safety instructions to avoid accidents, injury and damage.

#### Intended Purpose

- This product is intended for the measurement of human body temperature only. It is designed for private use.
- . The use of temperature readings for self-diagnosis is dangerous. Consult with a doctor for the interpretation of results. Self-diagnosis may lead to the worsening of medical
- This product is not intended for use by persons (including children) with limited physical. sensory or mental abilities or lacking experience and/or knowledge, unless they are supervised by a person responsible for their safety or have received and understand the instructions/ safety warnings for use of the product from this person in advance.

### Safety Warnings

- KEEP BATTERIES OUT OF REACH OF CHILDREN
- Swallowing may lead to serious injury in as little as 2 hours or death, due to chemical burns and potential perforation of the oesophagus.
- If you suspect your child has swallowed or inserted a button battery immediately call the 24-hour Poisons Information Centre on 13 11 26 (Australia) or 0800 764 766 (New Zealand) for fast, expert advice.



WARNING

- Examine devices and make sure the battery compartment is correctly secured, e.g. that the screw or other mechanical fastener is tightened. **DO NOT** use if compartment is not secure. Dispose of used button batteries immediately and safely. Flat batteries can still be dangerous
- Tell others about the risk associated with button batteries and how to keep their children safe.
- . Keep the packaging, product and its components out of reach of children. This product contains small parts, which are a **choking hazard** if swallowed.
- . A high or prolonged fever requires medical attention, especially for young children. Please contact a doctor.
- This product is for oral, rectal or underarm use only, **DO NOT** attempt to take temperatures at other sites, such as in the ear, as it may result in false readings and may lead to injury.
- Oral use is only for ages 2+.
- . Avoid using the product during or within 30 minutes after strenuous exercise: a shower, bath or sauna: using a cold compress; heavy alcohol consumption; smoking and food or drink intake.
- . DO NOT attempt measurements when the product is wet as inaccurate readings may result
- . DO NOT bite on the product.

- Avoid excessively bending the thermometric tip. DO NOT bend flexi tip (WDT505) more than 45 degrees. DO NOT bend the rigid tip (WFT404)
- DO NOT force the product into the rectum. Stop insertion and abort the measurement when
- DO NOT attempt rectal measurements on persons with rectal disorders.
- . DO NOT drop, step or place heavy objects on the product, as it is a precision instrument
- DO NOT store the product in the storage case when it is damp. Wipe it off with a clean dry soft cloth first
- DO NOT attempt to open or disassemble the product (other than the battery compartment)
- DO NOT place battery near extreme heat as it may explode.
- This product is equipped with a Zinc Manganese Alkaline or Lithium Manganese Button Battery. Failure to follow these instructions could cause the battery to leak, become hot, explode or ignite and cause injury and/or damage.
- DO NOT expose to temperatures over 55 °C.
- If battery leakage occurs and comes into contact with the skin or eyes, immediately rinse with plenty of water and seek medical advice.
- . Install the battery carefully with the polarity in the correct position.
- DO NOT use batteries after their expiration date.
- . Periodically check the battery to ensure it is in good working condition.
- Replace worn batteries with new ones immediately. Dispose of the old batteries according to current local regulations.
- Remove batteries carefully if the product is not going to be used for a long period of time.
- DO NOT use with other electrical or medical equipment simultaneously.
- DO NOT use within 30cm of a mobile device or a device that emits electromagnetic fields. as this may cause interference with the functioning of the products.
- DO NOT use or store the product in the following conditions:
- Near sources of heat, in hot water, direct sunlight or high temperature
- Moist or humid environments
- Dusty locations
- Near fire, flammable gas, chemicals or oxygen rich environments
- Locations easily subject to vibrations or shock or in a pressurized container
- In high frequency or strong magnetic field areas e.g. surgical equipment,
- MRI. CT. microwave oven

## Package Contents

- 1 x Thermometer
- 1 x Storage Case
- 1 x Battery (installed in product) 1 x Instruction Manual & Warranty Information

#### **Digital Thermometer Components** ON/OFF button Thermometric tip (0) Thermometric sensor — LCD screen Screw -Storage Case Battery compartment

# Before using the Digital Thermometer

- . Check the product is intact, DO NOT use if broken.
- Read the instruction manual thoroughly before taking the measurement.
- Allow product to rest for at least 30 minutes at room temperature (5°C 32°C).
- The user must remain in a stable environment for at least 5 minutes prior to taking a reading.
- Test the user's temperature when they are well so there is a comparison when they are unwell.
- Temperature readings are affected by many factors including physical exertion, the drinking
- of hot or cold beverages before measurement, as well as measuring technique. Avoid using the product during or within 30 minutes after exercise; a shower, bath or sauna;
- using a cold compress; heavy alcohol consumption; smoking and food or drink intake . Underarms must be clean and free of sweat, deodorant and scars, etc. for underarm use.
- Always make sure the thermometric sensor is clean before each use.
- . When measuring the temperature of an infant, note that infant body temperature is generally higher than adult body temperature and easily affected by external factors. For example, body temperature tends to increase after nursing or crying. Therefore, it is recommended to take measurements when the infant is calm.

# **Using the Digital Thermometer**

The use of temperature readings for self-diagnosis is dangerous. Consult with a doctor for the interpretation of results. Self-diagnosis may lead to the worsening of medical conditions.

### Switching the measuring unit between °C and °F

• While the product is OFF, press and hold the ON/OFF button (1) for approximately 2 seconds until the measuring unit changes. The product will automatically save your choice once you let go of the button and run a self-check before going into measurement mode.



# Taking a Temperature Measurement

• Note: The ambient temperature for use must be between 5-32°C. This product is a maximum thermometer, i.e. the displayed temperature can increase slightly if measurement continues after the beep. Holding the product too long may cause a higher ambient temperature reading of the probe. This could make the body temperature measurement lower than usual.

#### Before using, please disinfect the thermometric tip (refer to Cleaning section).

1. To turn on, press the (1) button next to the screen; a short beep will sound, indicating that the product is operational.



2. At the same time the product runs a selfcheck test, during which all the digital segments appear on the screen.


A default value of

37.0°C or 98.6°F will

display for 2 seconds.

The memory will show the last reading on the screen for 2 seconds

(last reading)

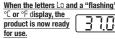

- Refer to Methods of measuring temperature section for placement instructions.
- Keep the user still during the measurement.
- 8. During the measurement, the °C or °F symbol "flashes" and a temperature will be displayed that will continue to change until a constant temperature is reached.
- 9. As soon as the constant temperature value is reached (approx. 10 seconds)\*, a beep will sound 10 times and the °C or °F symbol will stop flashing.
- 10. Once the beeping has stopped the measurement is completed. Remove the product from the measurement site.
- 11. The highest temperature measured appears on the screen.
- 12. Turn the product off by pressing the (1) button.
- 13. Although the product will turn off automatically after 10 minutes, to conserve power it is recommended to turn the product off manually after each reading.
- The reading will automatically be stored in the memory.
- 15. The measurement continues even after the beeping has stopped. In order to achieve better body temperature measurement results, it is recommend to keep the probe in mouth and rectum for an additional 2 minutes, or in armpit about for an additional 5 minutes after the beeping sound.
- Note: Result times may vary due to the individual and method used.

### Memory Function

• The last measured temperature will display on the screen for approximately 2 seconds after the self-check when the product is turned on.


# **Methods of Measuring Temperature**

Note: Measurement accuracy cannot be assured when the method used to measure the temperature is incorrect. If multiple measurements are taken over a period of time, it is important to use the same method of measurement each time so the results are comparable, i.e. do not interchange between oral, rectal or underarm measurements. If taking multiple measurements within a short period of time, allow the thermometer to return to room temperature prior to taking each measurement.

## Oral (ages 2+ only)

**Note: DO NOT** eat or drink anything before the measurement, or bite on the product.

- To ensure that reading is as accurate as possible, place the thermometric sensor to the left or right
  of the underneath/root of the tongue. Ensure the screen faces upwards.
- The thermometric sensor must have constant contact with the tissue during the reading and be placed under the tongue in one of the two heat pockets at the back. Press the tongue down to hold the product in place.
- Keep the mouth closed during the reading and breathe evenly through the nose, while holding onto the product to keep it from moving during measurement.
- There are different heat zones in the mouth. Generally, the oral temperature is 0.3°C to 0.8°C lower than the rectal temperature.

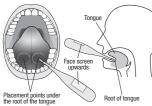

### Rectal

**Note:** This is the most accurate method from a medical point of view, because it comes closest to the core body temperature. It is commonly used for young children when it is difficult to take an oral or underarm temperature. **DO NOT** attempt rectal measurements on persons with rectal disorders.


- · Lubricate with a water-soluble gel. DO NOT use petroleum jelly.
- Gently insert the thermometric sensor no more than 2cm into the rectum. DO NOT force the
  product into the rectum. Stop insertion and abort the measurement when pain is present.

**Note:** The rectal method is the most accurate method for identifying the basal temperature and it is advised to extend the measuring time by 3 minutes after the beep.

## Underarm (axillary)

**Note:** Wipe off any underarm perspiration and deodorant prior to taking a reading under the arm.

- Place the thermometric sensor in the centre of the armpit. It must be touching the skin, remove clothing if necessary.
- Turn the screen so that it faces inwards.
- Hold the product in place by slightly applying pressure inwards towards the body.

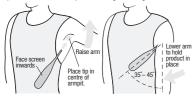

- . The product should be placed at an angle of 35-45 degrees in relation to the arm.
- When using with infants or young children, it is advised to gently hold their arm in place while taking the measurement.

Placing the product in the armpit provides a measurement of surface temperature that can fluctuate by around  $0.5^{\circ}\text{C}$  to  $1.5^{\circ}\text{C}$  from rectal temperature readings in adults.

Note: An exact reading cannot be obtained if, the armpits have been allowed to cool. If this is the case, we recommend extending the measuring time by around 5 minutes in order to obtain the most precise possible reading that corresponds as closely as possible to the core body temperature.

## Maintenance

#### Cleaning

- · Always clean/disinfect the product before and after use to avoid cross contamination.
- . Only clean with disinfectant (70% medical alcohol) using a damp cloth.
- Rinse only the thermometric tip under slow running cold water and then dry with a clean soft cloth.
- . DO NOT use harsh chemical cleaning agents.
- DO NOT immerse the product under water; rinse with hot water over 50°C; soak in alcohol
  for a long period of time; or use ultrasonic washing to clean the product as this can damage
  the electronics.

#### Storage

- . Wipe away any dirt and dust from the product
- Store the product in it's protective cover while not in use.
- DO NOT store the product in the protective cover when it is damp. Wipe it off with a clean dry soft cloth first.
- Store in a safe cool dry place, away from children as per Safety Warnings.
- . DO NOT place any heavy objects on the product.

## **Battery Replacement**

- When the low battery symbol is shown on the bottom right of the screen, please replace the battery.
- Using a small Phillips Head screwdriver size #00 or #000 to remove the safety screw from the back of the battery compartment cover.

Remove safety screw

METER WILDINGS
MITHEMATER
2010 107 NO.
TYLES WEEKSTER

compartment

Remove battery

compartment cover

cover

- Remove the battery compartment cover and gently pull out plastic circuit board with battery chamber approximately 1 cm.
- Use pointed object such as a pen to remove the old battery.
- Dispose of the old battery according to current local regulations.
- · Only use batteries specified for the product.
- Install the battery carefully with the polarity in the correct position with the positive(+)
  polarity facing towards the back of the thermometer.
- Slide battery chamber back into place.
- · Replace the battery compartment cover and ensure the safety screw is tightly secured.

# **Troubleshooting**

### A full troubleshooting guide can be found at www.jadavey.com.au

If the problem is not solved by the troubleshooting guide or you have any queries, please contact our service centre:

In Australia - 1800 807 464 In New Zealand - 0800 523 583

DO NOT attempt to open or repair the product yourself.

Note: The product is initially calibrated at the time of manufacture. If this product is used according to the instructions, re-calibration should not be required. However, if you feel that the product is out of specification, please contact us.

Power Type/Pating Zine Managenese Alkeline Putter Pattery / D41 1 EV DC)

## **Technical Data**

Power Type/Rating	Zinc Manganese Alkaline Button Battery (LR41 1.5V DC)
Battery Life	Approx. 200 hours
Thermometer Type	Maximum (Oral, Rectal & Underarm). Sensing unit = Thermistor.
Measurement Range	32.0 - 42.9°C / 89.6 - 109.2°F
Measurement Scale Increments	0.1°C / 0.1°F
Measurement Accuracy	±0.1°C (±0.2°F) during 35.5°C~42.0°C (95.9°F~107.6°F) at 18°C~28°C (64.4°F~82.4°F) ambient operating range
	±0.2°C (±0.4°F) for other measuring and ambient operating range
Operating Conditions	Temperature: 5 - 40°C Humidity: 15°C - 95°C RH (non-condensing) Atmospheric pressure: 700hPa to 1060hPa
Storage/Transport Conditions	Temperature: -20 - 55°C  Humidity: 15°C - 95°C RH (non-condensing)  Atmospheric pressure: 700hPa to 1060hPa
Dimensions	13.8cm x 2.2cm x 1.2cm (L x W x H)
Weight	Approx. 12g including battery
IP Rating	IP27 The first digit indicates the level of protection that the enclosure provides against access to any hazardous parts and the ingress of solid foreign particles.  2 = protection against particles >12.5mm The second digit indicates the level of protection that the enclosure provides against the harmful ingress of water.  7 = Protected from immersion between 15 cm and 1 m in depth.

Type BF Equipment  - Degree of protection against electric shock	
ABS + TPE plastic, Stainless Steel, Silica Gel, LCD Screen, Nickel Plated Iron Safety Screw	
Welcare Digital Thermometer, Storage Case, Battery (installed in product), Instruction Manual	



#### Manufactured by:

JOYTECH HEALTHCARE CO., LTD

No.365, Wuzhou Road, Yuhang Economic Development Zone, 311100, Hangzhou, China.

Distributed by: (New Zealand)
(Australia) JA DAVEY LTD
JA DAVEY PTY LTD
626 Lorimer Street 58 Richard Pearse Drive

Port Melbourne, VIC 3207 Mangere, Auckland 2022
Your Welcare Digital Thermometer comes with a 1 Year Limited Warranty.

See the enclosed Warranty Card for further information.

Register your warranty online at www.iadavev.com.au

For warranty repairs call:

In Australia - 1800 807 464 In New Zealand - 0800 523 583

This device complies with the provisions of the EC directive 93/42/EEC (Medical Device Directive) and the European Standard EN124703:2000-A1:2009, Clinical thermometers - Part 3: Performance of compact electrical thermometers (nonpredictive and predictive) with maximum device.

#### Important information regarding Electro Magnetic Compatibility (EMC)

Welcare Digital Thermometer manufactured by Joytech Healthcare Co., Ltd conforms to EN60601- 1-2:2015 Electro Magnetic Compatibility (EMC) standard.

Further documentation in accordance with this EMC standard is available at http://www.estore.jadavey.com.au

# Disposal



At the end of its life, please treat this product and batteries as electronic waste and dispose of responsibly according to current local regulations.

v10/20