

CoolGRIP™

Tubular Elastic Support Bandage



Size	Product code	Width (unstretched)	Limb circumference guide	Limb guide
A	36361527	4.5cm	10-12cm	Infant feet and arms
B	36361528	6.25cm	12-15cm	Small hands and arms
C	36361529	6.75cm	15-22cm	Medium arms and small ankles
D	36361530	7.75cm	22-32cm	Large arms, medium ankles and small knees
E	36361531	8.75cm	32-40cm	Large ankles, medium knees and small thighs
F	36361532	10cm	40-50cm	Large knees and medium thighs
G	36361533	12.5cm	50-60cm	Large thighs
H	36361538	14cm	60-67cm	Extra-large thighs



This tubular bandage must only be fitted under the instructions of a qualified health care practitioner. The bandage should be fitted at approximately 40-70% of maximum stretch. It must never be fitted at or near maximum stretch. Always ensure that the tubular bandage selected is not too tight for the limb or torso. In all instances, the bandage applied must not restrict blood flow or breathing, also allowing for any swelling of the limb or torso that may occur after the bandage is fitted. Do not wear the bandage whilst sleeping.

CoolFAST™

Tubular Retention Bandage



Colour code	Product code	Width (unstretched)	Width (maximum stretched, approx)	Limb circumference guide
RED	36361534	3.5cm	12cm	10-16cm child hands and arms
GREEN	36361535	5cm	18cm	14-24cm child legs and adult arms
BLUE	36361536	7.5cm	27cm	21-36cm adult arms, lower legs, knees and small thighs
Yellow	36361537	11cm	36cm	30-50cm adult knees and thighs



This tubular bandage must only be fitted under the instructions of a qualified health care practitioner. The bandage should be fitted at approximately 40-70% of maximum stretch. It must never be fitted at or near maximum stretch. Always ensure that the tubular bandage selected is not too tight for the limb or torso. In all instances, the bandage applied must not restrict blood flow or breathing, also allowing for any swelling of the limb or torso that may occur after the bandage is fitted.

Smith & Nephew Pty Ltd

Australia

T 61 2 9857 3999 F 61 2 9857 3900

www.smith-nephew.com/australia

Customer Service

T 13 13 60 F 1800 671 000

New Zealand

T 64 9 820 2840 F 64 9 820 2841

www.smith-nephew.com/new-zealand

Customer Service

T 0800 807 663 F 0800 263 222